

Titanium Dual Pyramid Receiver Instructions

WHAT'S IN THE BOX

One of the following:

- Titanium Dual Pyramid Receiver, 32 mm (FND-228032)
- Titanium Dual Pyramid Receiver, 45 mm (FND-228045)
- Titanium Dual Pyramid Receiver, 60 mm (FND-228060)
- Titanium Dual Pyramid Receiver, 75 mm (FND-228075)

Instructions

INSTALLATION

Connect the Titanium Dual Pyramid Receiver to the pyramid components of choice. Once alignment has been achieved, apply Loctite 242 (or equivalent) to all pyramid setscrews and tighten the setscrews to 12 ft-lbs (16 Nm).

DIFFERENT LENGTHS AVAILABLE

There are four available lengths of Titanium Dual Pyramid Receivers: 32 mm, 45 mm, 60 mm, and 75 mm. If the length that was initially ordered is inappropriate, please contact WillowWood to order one of the other lengths.

EXPOSURE TO LIQUIDS

Though this WillowWood component is resistant to fresh water splashes, light sprays, and sanitizing with damp cloths, it is not rated for prolonged exposure to moisture or liquids such as partial or total immersion in water (as in swimming, bathing, and showering). This product is also not rated for any exposure to pressure sprays, salt water, chlorinated water, soap, or any liquid with chemical content.

WARRANTY

The warranty for the Titanium Dual Pyramid Receiver is one year from date of invoice, provided that the component is selected according to the following criteria:

Note: The weights listed below are adjusted body weights. Adjusted body weight is defined as the weight of the amputee plus any loads normally or routinely carried by the amputee.

	Level 1	Level 2	Level 3	Level 4
Titanium Dual Pyramid Receiver (Product No. FND-2280XX)	350 lbs 160 kg	350 lbs 160 kg	350 lbs 160 kg	300 lbs 135 kg

Level 4: Has the ability or potential for prosthetic ambulation that exceeds basic ambulation skills, exhibiting high impact, stress, or energy levels. Typical of the prosthetic demands of the child, active adult, or athlete.

Use of a Titanium Dual Pyramid Receiver for amputees who do not meet the above criteria or who engage in extremely high and abusive activities is against WillowWood's recommendations and will void the one-year warranty. "Extremely high and abusive activities" are defined as activities such as skydiving, karate, and judo; activities that could result in injury to an individual's natural limbs; and activities that expose the prosthesis to corrosives such as salt water.

WARRANTY DISCLAIMER

WillowWood warrants that each product manufactured will, at the time of delivery, be of workmanlike quality and substantially free of defects. WILLOWWOOD MAKES NO OTHER WARRANTY, IMPLIED, OR EXPRESSED, AND MAKES NO WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. This warranty shall terminate immediately upon an action to combine our products with other materials or in any manner to change the nature of our products. The sole remedy is replacement of the products or credit for the products. Willow Wood's liability shall not exceed the purchase price of the product. WillowWood shall not be liable for any indirect, incidental, or consequential damage.

WILLOWWOOD RETENTION OF RIGHTS

WillowWood retains all intellectual property rights reflected or incorporated in its physical products, regardless of the transfer of the physical products to another party or parties.

WillowWood®

The Ohio Willow Wood Company
15441 Scioto Darby Road
Mt. Sterling, OH 43143
phone 740.869.3377 / 800.848.4930
fax 740.869.4374 www.willowwoodco.com



Ohio Willow Wood Company B.V.
Keizersgracht 62/64
1015 CS Amsterdam
The Netherlands



PN-1455-E 14 DEC 2018