

Foundation System 4-Hole Pyramid Adapter/Receiver Instructions

WHAT'S IN THE BOX

One of the following:

- Composite 4-Hole Pyramid Adapter (FND-994100)
- Composite 4-Hole Pyramid Adapter, 5 mm Offset (FND-994105)
- Composite 4-Hole Pyramid Adapter, 10 mm Offset (FND-994110)
- Titanium 4-Hole Pyramid Adapter, 20 mm Offset (FND-224120)
- Titanium 4-Hole Pyramid Adapter (FND-227014)
- Titanium 4-Hole Pyramid Receiver (FND-227004)

Instructions

INSTALLATION

1. Fabricate a socket with a 4-hole pattern at the distal end using a WillowWood Laminated Socket Attachment Block (Part No. 700-250 or 700-200), an Alpha Lock (Part No. 700-AIS480), a Suction Pro (Part No. 700-SP100), a Lamination Lock (Part No. 700-AL600), a Locking Lanyard (Part No. 700-AIS100), or equivalent 4-hole socket component.

Note: Make sure that the distal surface of the socket has a flat surface to provide a secure attachment for the adapter.

2. Attach a 4-Hole Pyramid Adapter/Receiver to the 4-hole socket component using four 6 mm flathead screws of the appropriate length and strength. Apply Loctite 242 (or equivalent) and tighten the screws to 9 ft-lbs (12 Nm).
3. Install the mating pyramid component of your choice onto the Adapter/Receiver. Once alignment has been achieved, follow the torque and threadlocker specifications provided by the manufacturer of the pyramid receiver. If using the Titanium 4-Hole Pyramid Receiver (FND-227004), apply Loctite 242 (or equivalent) to the setscrews and tighten the setscrews to 12 ft-lbs (16 Nm).

EXPOSURE TO LIQUIDS

Though this WillowWood component is resistant to fresh water splashes, light sprays, and sanitizing with damp cloths, it is not rated for prolonged exposure to moisture or liquids such as partial or total immersion in water (as in swimming, bathing, and showering). This product is also not rated for any exposure to pressure sprays, salt water, chlorinated water, soap, or any liquid with chemical content.

WARRANTY

The warranty for 4-Hole Pyramid Adapters/Receivers is one year from date of invoice. Use of a 4-Hole Pyramid Adapter/Receiver for amputees whose adjusted body weight is greater than 250 lbs (115 kg) or who engage in extremely high and abusive activities is against WillowWood's recommendations and will void the one year warranty. Adjusted body weight is defined as the weight of the amputee plus any loads normally or routinely carried by the amputee. "Extremely high and abusive activities" are defined as activities such as skydiving, karate, and judo; activities that could result in injury to an individual's natural limbs; and activities that expose the prosthesis to corrosives such as salt water.

WARRANTY DISCLAIMER

WillowWood warrants that each product manufactured will, at the time of delivery, be of workmanlike quality and substantially free of defects. WILLOWWOOD MAKES NO OTHER WARRANTY, IMPLIED, OR EXPRESSED, AND MAKES NO WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. This warranty shall terminate immediately upon an action to combine our products with other materials or in any manner to change the nature of our products. The sole remedy is replacement of the products or credit for the products. WillowWood's liability shall not exceed the purchase price of the product. WillowWood shall not be liable for any indirect, incidental, or consequential damage.

WILLOWWOOD RETENTION OF RIGHTS

WillowWood retains all intellectual property rights reflected or incorporated in its physical products, regardless of the transfer of the physical products to another party or parties.

WillowWood®

The Ohio Willow Wood Company
15441 Scioto Darby Road
Mt. Sterling, OH 43143
phone 740.869.3377 / 800.848.4930
fax 740.869.4374 www.willowwoodco.com



Ohio Willow Wood Company B.V.
Keizersgracht 62/64
1015 CS Amsterdam
The Netherlands



PN-1530-H 14 DEC 2018