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Before putting on the sleeve, cover open wounds and non-intact skin with a gauze patch.

Inspect your sleeve after each day of use. Unusual wear may indicate changes in the prosthetic fit. Call your prosthetist immediately if you notice unusual wear.

DONNING

Do not apply any creams, lotion, powders, or medicines to the limb. These products could damage the sleeve gel.

Orient the Alpha Basic Sleeve so that the partial fabric cover on the interior of the sleeve covers the brim area of the socket, especially in the anterior area. The exposed gel area above and below the partial fabric cover should contact the skin (or liner) and socket, respectively.

If you are sweating heavily, remove the sleeve and liner and dry off the limb and liner as needed.

CLEANING

We recommend keeping the surface of the gel clean and clear of debris. After cleaning, hang the sleeve up to dry.

Do not clean the sleeve with harsh detergents or antimicrobial soap.

WEEKLY DISINFECTION

If the sleeve is worn so that the gel portion of the sleeve touches the skin, then the area of the gel that touches the skin should be disinfected once a week.

1. Place a small amount of ethyl or isopropyl alcohol on a soft, clean cloth.
2. Lightly wipe the gel side of the sleeve with the cloth.
3. Rinse the alcohol from the sleeve with water.
4. Hang the sleeve up to dry.

Note: Do not soak the sleeve in the ethyl or isopropyl alcohol. Extended contact with large amounts of ethyl or isopropyl alcohol will stiffen the sleeve.

STORAGE

When the prosthesis is not being worn for a few days, the distal portion of the sleeve may be left on the socket.

Do NOT leave the sleeve on the socket with the top half of the sleeve reflected back onto the socket, or the sleeve will stretch out.

If the prosthesis is not to be worn for five days or more, the sleeve should be removed, cleaned, and stored in a clean, dry place.

If the sleeve is left on the prosthesis, the proximal edge of the sleeve must be positioned as shown and must NOT be reflected back onto the socket.

Alpha Products can melt or burn if exposed to high temperatures or flame. Do not expose your Alpha product to these conditions.

This Alpha product is intended for use on a single patient. Use of the product with multiple patients could lead to cross contamination between patients.

Warning: To avoid danger of suffocation, keep this product away from babies or children.